

Plastodur

Application:

Improves the effectiveness of isolating agents and prevents abrasion (e.g. due to contacts with antagonists during articulation movements).

Mode of application:

- Apply 1-2 thin layers with brush as needed without thinner.
- Scratch-resistant after 15 min.
- Hard after 30 min.

Protective measures:

Ensure sufficient fresh air supply.

Properties:

Hardens rapidly, scratch-resistant, water-repellent.

Shel life:

2 years

Storage:

- At constant temperature not above 35°C.
- Do not expose to direct sunlight.
- Protect from frost.

Attention:

Do not empty into the sewage system.

Manufacturer:

DFS-DIAMON GmbH * Ländenstraße 1 * D-93339 Riedenburg * +49 9442 91890

Rev. 11/14