

Conductive Lacquer GALVANET

Field of application:

Electroplating / Galvano technique

Handling instructions:

Shake well before use!

Keep the bottle well sealed after use!

Degrease metal surfaces before application (e.g. with alcohol). In case of application on plaster ensure completely dried out surfaces. We recommend using micro wave drying if necessary. Apply **one even covering layer** of lacquer on the entire surface as thin as possible. Avoid any touching-up and puddles otherwise the silver layer will be too thick! Let it dry completely. GALVANET dries quickly, nevertheless we recommend observing a drying time of 5 min before proceeding with further work.

When the electroplating process has finished, remove the object from the electric current and dissolve the plaster by a plaster solvent in an ultrasonic bath. In case of direct electroplating on a metal surface, remove the fine gold coping carefully from the primary crown. The silver lacquer can be easily dissolved from the gold coping by nitric acid of 30% in an ultrasonic bath.

Protective measures:

- Keep away from combustible material
- Avoid skin contact
- Do not use water for fire extinguishing
- Take care for sufficient room ventilation, do not breathe vapors
- Wear suitable gloves
- We recommend wearing safety goggles
- Special waste disposal. Do not empty into drains

Properties:

Flammable

Storage:

Store cool or at ambient room temperature

Manufacturer:

DFS-DIAMON GmbH * Ländenstraße 1 * D-93339 Riedenburg * +49 9442 91890

Rev. 09/14