# Instructions for use

Page 1 / 1

## Isofix



# **Application:**

Isolation of model dies during waxwork in the crown and bridgework

## **Mode of application:**

Coat the die thinly once with the brush integrated into the bottle cap and let sit for app. 30 sec. Remove excessive liquid softly by blowing air. Apply wax or dipping wax as usual. Close bottle firmly after use.

#### **Quality:**

Isofix forms a thin microfilm of about 3-5µ. Compatible with dental modelling waxes. Facilitates easy lifting of waxwork without gaps or distortions. Avoids formation of flow marks on the inner side of the crown. Does not react with investment. Solvent-free and water soluble.

# Storage:

Store firmly closed

### **Disposal:**

With general lab waste (glass)

# **Manufacturer:**

DFS-DIAMON GmbH ★ Ländenstraße 1 ★ D-93339 Riedenburg ★ +49 9442 91890

Rev. 02/16