

Kera-Sep Spray

Application:

Separating spray for ceramics. To insulate models from applied ceramic mass. In conjunction with silicone occlusal surface modelling help Okkluflex (DFS-Diamon GmbH).

Mode of application:

Shake well before use.

Wait for app. 1 min after spraying from a distance of 5-8 cm, the spray a second time. Apply only short bursts, avoid puddles, e.g. inside alveolae.

Application by brush is also possible.

Quality:

Kera-Sep Spray safely insulates model surfaces from applied ceramics and allows easy lifting of modellings for the ensuing firing. No risk of cross-contamination as with brush bottles. Liquid burns without residue.

Risks:

Avoid inhaling spray vapor.

Storage:

- Container is under pressure.
- Do not expose to direct sunlight.
- Protect from heat in excess of 50°C.

Disposal:

Together with general lab waste (metal can).

Manufacturer:

DFS-DIAMON GmbH * Ländenstraße 1 * D-93339 Riedenburg * +49 9442 91890

Rev. 11/14