

Fluxur

Product name:

Fluxur, flux material for all dental alloys

Composition:

Complex calcium- hydroxo-fluoroborate

Application:

Crown-& bridgework, partial framework

Mode of application:

Prepare the area to be soldered to be metallically bare. Prior to heating by flame apply sufficient Fluxur on the area to be soldered. Keep heating until red heat, if necessary add Fluxur. Apply solder, allow it to inject into the solder-jointing, briefly heat afterwards.

After slow cooling down, mechanically remove the vitrified flux material (grinding, sand blasting), do not use acid-etching or pickling agents.

We recommend using rotary wire brushes (DFS-Diamon order no. 60014).

Manufacturer:

DFS-DIAMON GmbH * Ländenstraße 1 * D-93339 Riedenburg * +49 9442 91890

Rev. 11/13