

Instructions for use

Copyright by DFS-Diamon GmbH

Page 1 / 1



Clean-Orange (Orange Cleaner)

Date of issue: 18.11.2014

Last revision date:



*Example

Application

Surface cleaning of furniture and stainless steel surfaces in dental clinics and labs. Cleaning of impression trays. Acrylic surfaces may show slight damage when using Clean-Orange.

Mode of Application

Mix with water depending on the intensity of dirt to be cleaned.

Qualiy

Removes residue of silicone, alginates, zinc-eugenol pastes and blood. Not suitable for use in ultrasonic cleaners.

Risks

Do not drink or swallow, please consult a doctor if the liquid is swallowed. Avoid contact to skin and eyes, may cause skin irritations.

Disposal

Do not pour into the drain the pure condition.

Rev. 11/14



DFS-Diamon GmbH
Ländenstraße 1
93339 Riedenburg
Germany